

Titan SGS™ Standard
Staple-Line Reinforcement
featuring GORE® SEAMGUARD®
Bioabsorbable Staple Line
Reinforcement Material



Titan SGS™ Stapler

Titan SGS™ Standard Staple-Line Reinforcement
featuring GORE® SEAMGUARD® Bioabsorbable
Staple Line Reinforcement Material

Titan SGS™ Standard Staple-Line Reinforcement and Titan SGS™ Stapler

The Titan SGS™ Stapler, the only purpose-built stapler for gastric tissue with specific indication for sleeve pouch creation in bariatric procedures,¹ is now paired with GORE® SEAMGUARD® Reinforcement Material. This is the only*² staple line reinforcement (SLR) material proven to significantly reduce leaks in sleeve gastrectomy procedures.³

Paired for use in sleeve gastrectomy


Just as not all bariatric staplers are the same, not all staple line reinforcements are the same. The Titan SGS™ Stapler is a 23cm, single-fire stapler designed to deliver more consistent and symmetrical sleeve anatomy by enabling the Standard Sleeve™ Technique, an anatomy-based approach to sleeve gastrectomy.¹ It is now available with Titan SGS™ Standard Staple-Line Reinforcement featuring GORE® SEAMGUARD® Reinforcement Material.

For more than 20 years† GORE® SEAMGUARD® Bioabsorbable Staple Line Reinforcement Material has created an unrivaled legacy of clinical performance.³⁻⁷ Two decades of robust clinical literature detail GORE® SEAMGUARD® Reinforcement Material leak protection and bleeding reduction.⁷

* Considering all systematic review and meta-analysis of published articles only that distinguish between types of staple line reinforcement as of January 2024.

† First commercial marketing authorization, U.S. Food and Drug Administration in April 2003.

Strengths of GORE® SEAMGUARD® Reinforcement Material

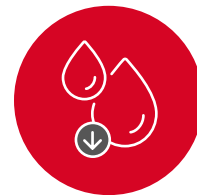

0.39% LEAK RATE
in the U.S.³



5 MILLION+
devices implanted²



85+ CLINICAL
STUDIES
published and
peer-reviewed²

Titan SGS™ Standard Staple-Line Reinforcement featuring GORE® SEAMGUARD® Bioabsorbable Staple Line Reinforcement Material

A Closer Look at GORE® SEAMGUARD® Reinforcement Material



Designed to protect against leaks

GORE® SEAMGUARD® Reinforcement Material has a significantly lower leak rate compared to other laparoscopic sleeve gastrectomy methods as shown in a systematic review consisting of 148 papers and more than 40,600 patients.³

A comparison of laparoscopic sleeve gastrectomy leak rate percentages in the U.S. among five staple line reinforcement (SLR) types:³

GORE® SEAMGUARD® Reinforcement Material	0.39%
SUTURE reinforcement by over sewing alone	0.7%
NO SLR no reinforcement	1.3%
SEALANT reinforcement with tissue sealant	1.8%
BPS reinforcement with nonabsorbable bovine pericardial strips	1.5%

Engineered to protect against bleeding and reduce bleeding complications

Randomized prospective data has shown that GORE® SEAMGUARD® Reinforcement significantly reduces bleeding in gastric bypass procedures.^{5,6}

GORE® SEAMGUARD® Reinforcement Material provides a matrix that blood can infiltrate.⁸ Platelets within the patient's blood stick together and create a clot to decrease bleeding. The material:

- Conforms to the staple line anatomy⁸
- Reduces blood flow by applying pressure⁸
- Provides a matrix for blood to start clotting⁸

Potential to reduce overall postoperative complication costs

GORE® SEAMGUARD® Reinforcement Material provides reliable protection from surgical complications associated with leaks and bleeding following the laparoscopic sleeve gastrectomy, along with reduced reoperations and readmissions.^{3,7,9}

- 1 510(k) No. K210278. The Titan SGS linear cutter is intended for longitudinal transection and resection of gastric tissue for sleeve gastrectomy pouch creation. 2021. Database Product Search (March 2023, Product Code: GDW).
- 2 Data on file 2022; W. L. GORE & Associates, Inc; Flagstaff, AZ.
- 3 Gagner M, Kemmeter P. Comparison of laparoscopic sleeve gastrectomy leak rates in five staple-line reinforcement options: a systematic review. *Surgical Endoscopy* 2020;34(1):396-407.
- 4 Gayrel X, Loureiro M, Skalli EM, Dutot C, Mercier G, Nocca D. Clinical and economic evaluation of absorbable staple line buttressing in sleeve gastrectomy in high-risk patients. *Obesity Surgery* 2016;26(8):1710-1716.
- 5 Miller KA, Pump A. Use of bioabsorbable staple reinforcement material in gastric bypass: a prospective randomized clinical trial. *Surgery for Obesity & Related Diseases* 2007;3(4):417-422.
- 6 Nguyen NT, Longoria M, Welbourne S, Sabio A, Wilson SE. Glycolide copolymer staple-line reinforcement reduces staple site bleeding during laparoscopic gastric bypass. A prospective randomized trial. *Archives of Surgery* 2005;140(8):773-778.
- 7 W. L. Gore & Associates, Inc. Clinical Performance with Staple Line Reinforcement. Scientific Literature Analysis (n = 8142 patients). Flagstaff, AZ: W. L. Gore & Associates, Inc; 2021. [Literature summary]. 2161018-EN.
- 8 GORE Medical. (n.d.). Bleeding - Protects against bleeds. GORE Medical GORE SEAMGUARD Bioabsorbable Staple Line Reinforcement Data Review – Bleeding. <https://www.goremedical.com/products/seamguard---bleeding>. Accessed 30 January 2024.
- 9 Zambelli-Weiner A, Brooks E, Brolin R, Bour ES. Total charges for postoperative leak following laparoscopic sleeve gastrectomy. Presented at Obesity Week 2013: The American Society for Metabolic and Bariatric Surgery and the Obesity Society Joint Annual Scientific Meeting; November 11-16, 2013; Atlanta, GA. A-305-P.

For more information about the Titan SGS™ Stapler and Titan SGS™ Standard Staple-Line Reinforcement, visit standardbariatrics.com/titansgs/slr

For more about GORE® SEAMGUARD® Reinforcement Material, visit goremedical.com/products/seamguard

To place an order: orders@standardbariatrics.com

Disclaimer: Before using any medical device review all relevant package inserts with particular attention to the indications, contraindications, warnings and precautions, and steps for use of the device.

GORE and SEAMGUARD are trademarks of W. L. Gore & Associates. Teleflex, the Teleflex logo, Standard Bariatrics, Standard Sleeve, and Titan SGS are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. All other trademarks are the property of their respective owners.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 2/2024. © 2024 Teleflex Incorporated. All rights reserved.

Workfront project code: 231228951-EN. CC21009.A · Rev A.

